

Instant-View

Instant-View Saliva Drug Test



DRUG TESTING KITS

- Simple, one-step collection process
- Results in 10 minutes
- Test for five drugs, including benzodiazepines

ALCO **PRO**
DRUG AND ALCOHOL TESTING PRODUCTS

1-800-227-9890

www.alcopro.com

Instant-View Saliva Drug Test



Simple, One-step Collection Process

The Instant-View saliva drug test kit builds the saliva collector into the test device, eliminating the messy process of transferring saliva from a separate collector onto the test device. The user places the end of the device with the collector pad into the mouth until the flow of saliva is visible in the results window. The operator then removes the device and waits for the lines to develop to indicate the presence or absence of drugs.

Results in 10 minutes

The Instant-View saliva drug test develops a line next to the indicator for each drug if that drug is not present in the urine specimen. If the drug is present in the urine specimen at or above the cut-off level for that drug, the presence of the drug prevents a line from forming. The absence of a line indicates the presence of drugs. The operator reads the Instant-View saliva drug test ten minutes after removing from the subject's mouth.

Detects Five Drugs of Abuse

The Instant-View saliva drug test detects five of the most commonly abused drugs: marijuana (THC), opiates, cocaine, benzodiazepines, and amphetamines. Because the window of detection for benzodiazepines in saliva is one to three days (compared to several weeks for urine), a positive saliva test for benzodiazepines indicates recent drug use.

SPECIFICATIONS

Test Medium: Saliva

Format: Integrated saliva collector

Cut-off levels

AMP (amphetamine)	50 ng/ml
BZD (benzodiazepines)	20 ng/ml
COC (cocaine)	20 ng/ml
OPI (opiates)	40 ng/ml
THC (marijuana)	12 ng/ml



Subject holds device in mouth until flow of saliva is visible in results window.



Read results five minutes after the control line develops.

Instant-View Saliva Drug Test

Item	Description	1-24	Box of 25 Tests
8VS54	THC, opi, coc, benzo, amp	\$13.95 @	\$323.75 (\$12.95@)

For Information & Orders Call: **1-800-227-9890**
 Order 24 Hours A Day At: **www.alcopro.com**
 Fax Orders To: **1-800-655-5834**

ALCO PRO
 DRUG AND ALCOHOL TESTING PRODUCTS