

# ABT

## Alcohol Breath Tube



### THE ALCOHOL BREATH TUBE INSTRUMENT

after drinking or smoking-or... warm water.  
from both ends of tube and di... active crystals. Save plugs for str...  
with full continuous deep bre...  
to end of tube. Allow air from... h tube for exactly one minute  
and cap tube with plugs. ... en within five minutes if alcoh...

### Reading Alcohol Concentration

read like a thermometer... an color after five m...

and green;  
ands green;  
bands green;

acturer or its agents... and with respect to  
s product. This prod... centration. Decisions  
his product by any p... reach of children.

ALCOPRO... 10954 • K... 37939

### ALCOHOL TESTING KITS

- Semi-quantitative results to 0.15 and 0.08 BrAC
- Accurate to +/- 0.02 BrAC
- Balloon ensures proper breath sample



1-800-227-9890

www.alcopro.com

# Alcohol Breath Tube

## Semi-quantitative Results

The Alcohol Breath Tube is one of the only disposable breath test devices that estimates alcohol concentrations. Alcohol in the breath causes yellow crystals to change to a green color. Higher concentrations of alcohol cause the green color to move higher up the tube. The user reads the result at the highest level of color change, much like reading a thermometer.

## Accurate to +/- 0.02 BrAC

The operator uses the % BrAC scale on the device to interpret breath test results. An independent study comparing the Alcohol Breath Tube to an evidential breath test device showed the Alcohol Breath Tube to be accurate to +/- 0.02 BrAC.

## Balloon Ensures Proper Breath Sample

Breath alcohol tests require a subject to deliver an adequate amount of "deep lung" breath to get an accurate estimate of intoxication level. The Alcohol Breath Tube uses a balloon to capture the subject's breath to ensure that each subject delivers the same amount of deep lung breath.

## Quick Results

Results are available within two minutes.

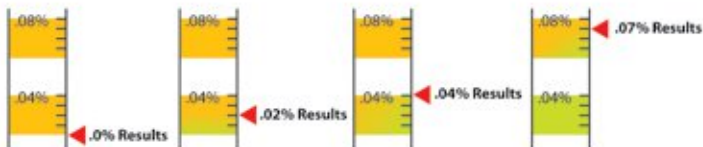


Illustration of various results



## SPECIFICATIONS

Test Medium: breath

Test principle: dichromate chemical, length of stain

Reading Time: 1-2 minutes

Results: semi quantitative

Measurement Range: ABT-08: 0.02% - 0.08% , ABT-15: 0.02% - 0.15% BrAC

Shelf life: 18 months from date of manufacture



### STEP 1

The user first pours out the blue preservative crystals.



### STEP 2

The subject then inflates the balloon attached to a special mouthpiece.



### STEP 3

The user attaches the balloon to the test and allows air to flow through the test for one minute.

## Alcohol Breath Tube

Item	Description	1-25	Box of 25
AAB0820	Alcohol Breath Tube (.08%)	\$2.19	\$48.75 (\$1.95@)
AAB1520	Alcohol Breath Tube (.15%)	\$2.19	\$48.75 (\$1.95@)

For Information & Orders Call: **1-800-227-9890**  
 Order 24 Hours A Day At: **www.alcopro.com**  
 Fax Orders To: **1-800-655-5834**

**ALCOPRO**  
 DRUG AND ALCOHOL TESTING PRODUCTS